



STOP

INOGEN ONE BATTERY CARE AND MAINTENANCE INSTRUCTIONS.

Please follow these important guidelines to maximize battery performance and life:

1. Remove battery from the Inogen One while not in use.
2. Store battery in a cool, dry place. Store with a charge of 40-50%.
3. Once per month: Fully charge the battery then discharge to 10% and charge back to full.
4. If using multiple batteries, make sure that each battery is labeled (1, 2, 3 or A, B, C, etc) and rotated on a regular basis. Batteries should not be left dormant for extended periods of time.

Note: Non-compliance with these guidelines will reduce battery life and may void your warranty.